

- ✓ Fast results only in 10 minutes
- ✓ No special equipment or training required
- ✓ All-in-one package

Instructions For Use

You must follow the test directions carefully to get an accurate result.

Each test can only be used once.

This test must be used within 5 days of when you first experienced symptoms.

Test Contents

- 1 TEST CASSETTE
- 1 STERILE ANTERIOR NASAL SWAB
- 1 EXTRACTION VIAL
- 1 EXTRACTION VIAL CAP




01.

Wash your hands thoroughly for at least 20 seconds before the test.



02.

Unpack the test components.

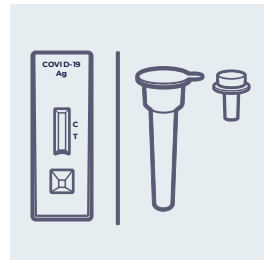


03.

Remove the test cassette and place it on a flat, clean surface.



04.


Locate the extraction vial and gently peel off the aluminium foil seal, being sure to keep the vial upright. The tray (available for pack sizes of 1 and 2 tests only) can assist with keeping the vial upright.

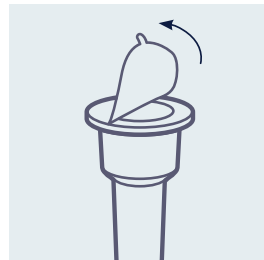

05.

Locate a nasal swab and remove from the pouch.


Be careful not to touch the swab tip.

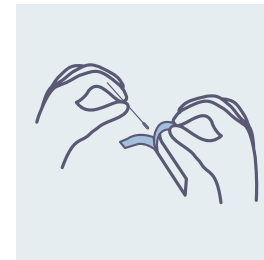

06.

Gently insert the swab up to 2cm (3/4 inch) into the LEFT nostril. Then, slowly rotate the swab 5 times in a circular path for 15 seconds.


Gently remove the swab from the LEFT nostril. Repeat this step for the RIGHT nostril.

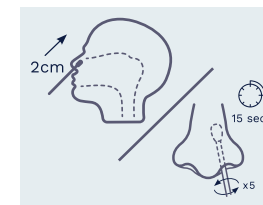

07.

Place the swab into the extraction vial.

Rotate the swab vigorously at least 5 times.

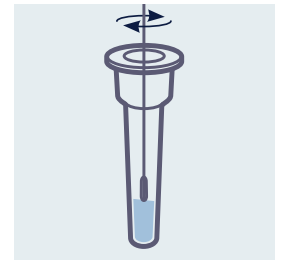


- ✓ Fast results only in 10 minutes
- ✓ No special equipment or training required
- ✓ All-in-one package

Instructions For Use (continued)


08.

Remove the swab by rotating against the extraction vial while squeezing the sides of the vial to release the liquid from the swab.

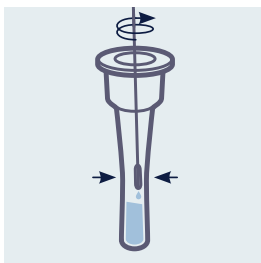

09.

Close the vial by pushing the cap firmly onto the vial.

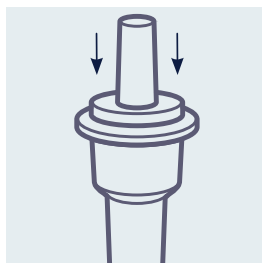


10.

With your finger, mix thoroughly by flicking the bottom of the vial.




11.

Invert the extraction vial and hold the sample vertically above the sample well. Squeeze the vial gently. Allow THREE (3) drops of sample to fall into the sample well.

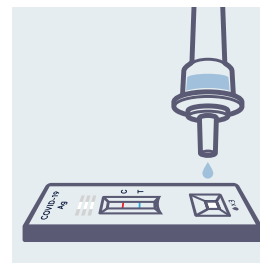

12.

Read the result at 10 minutes. The test result should not be read after 15 minutes.

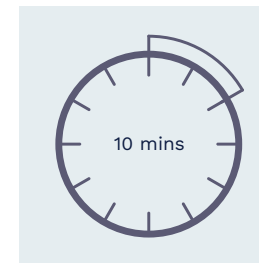

Disposal.

Carefully wrap the used test kit components and swab samples and dispose in normal household waste.




Reading The Results

Make sure you wait the full 10 minutes.

NOTE: The test results should be read at 10 minutes after the sample application. The test result must be read before 15 minutes as test results after 15 minutes may not be accurate.


COVID-19 Detected (Positive)

One red-coloured line next to "C" and one blue-coloured line next to "T" indicates COVID-19 positive result.

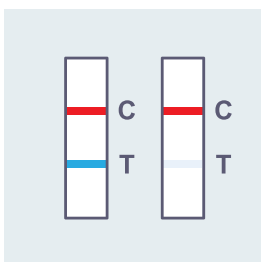

COVID-19 Not Detected (Negative)


One red-coloured line only next to "C" indicates a negative result. Re-test in 24-48 hours if your first test result is negative.

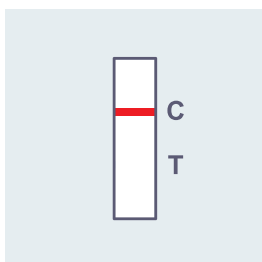

Invalid

If a red-coloured line in the control "C" is not visible, the result is invalid.

Re-test with a COVID-19 test may be needed.

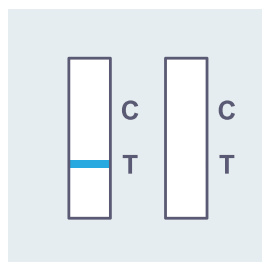

A positive test result indicates that antigens from SARS-CoV-2 were detected, and you are likely to be infected and presumed to be contagious. You must immediately:

Undergo a laboratory PCR test at a COVID-19 respiratory clinic or contact your doctor and they will arrange the test for you.

COVID-19 respiratory clinics are dedicated health centres located around the country, focusing on testing people who may have COVID.

CareStart™ Mobile Application

Scan the QR code for further information on how to complete the CareStart™ COVID-19 Antigen Home test.



SCAN ME